

FAST  
TEST



# Canine

**LYM Ab - ANA Ab**

**EHR Ab - CHW Ag**


BF-CLYAEH-10  
10 TESTS

# Result Interpretation



## Negative Result

(“C”)Line: Dark & (“T”) Line: Absent  
Absence of LYM Ab, ANA Ab, EHR Ab, and CHW Ag in the sample.  
Or undetectable levels of LYM Ab, ANA Ab, EHR Ab, and CHW Ag in the sample.

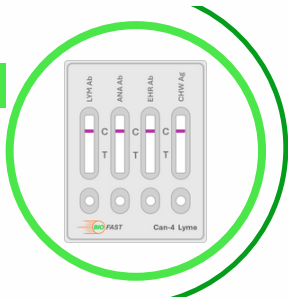

## Positive Result

(“C”)Line: Dark & (“T”) Line: Dark  
Presence of LYM Ab, ANA Ab, EHR Ab, and CHW Ag in the sample.

## Invalid Result

(“C”)Line: Absent & (“T”) Line: Absent  
(“C”)Line: Absent & (“T”) Line: Dark  
The sample should be re-tested.

