

FAST
TEST



Canine CPV Antibody

BF-CPABT-10
10 TESTS





Result Interpretation



Negative Result

("C")Line: Dark & ("T") Line: Absent
Absence of CPV antibodies in the sample.
Or undetectable levels of CPV antibodies in the sample.

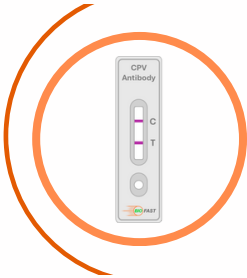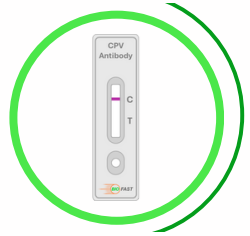

Positive Result

("C")Line: Dark & ("T") Line: Dark
Presence of CPV antibodies in the sample.

Invalid Result

("C")Line: Absent & ("T") Line: Absent
("C")Line: Absent & ("T") Line: Dark
The sample should be re-tested.

