

FAST  
TEST



# Canine CHW Antigen



BF-CHWAG-10  
10 TESTS

# Result Interpretation



## Negative Result

("C")Line: Dark & ("T") Line: Absent  
Absence of Heartworm in the sample.  
Or undetectable levels of Heartworm charge in the sample.

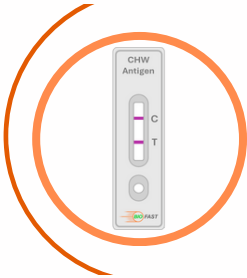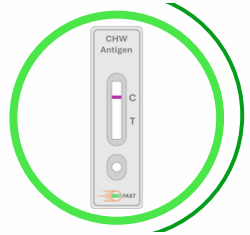

## Positive Result

("C")Line: Dark & ("T") Line: Dark  
Presence of Heartworm in the sample.

## Invalid Result

("C")Line: Absent & ("T") Line: Absent  
("C")Line: Absent & ("T") Line: Dark  
The sample should be re-tested.

