

FAST
TEST



Feline FeLV Antigen



BF-FLAG-10
10 TESTS

Result Interpretation



Negative Result

("C")Line: Dark & ("T") Line: Absent
Absence of FeLV in the sample.
Or undetectable levels of FeLV in the sample.

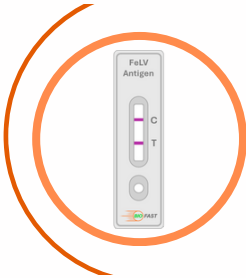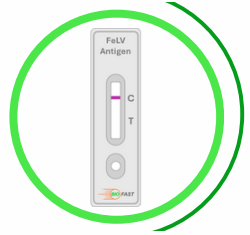

Positive Result

("C")Line: Dark & ("T") Line: Dark
Presence of FeLV in the sample.

Invalid Result

("C")Line: Absent & ("T") Line: Absent
("C")Line: Absent & ("T") Line: Dark
The sample should be re-tested.

